


ACCOMMODATE EVEN LONGER LOUPES WITH THE **ProKnee** ^{SINCE 1989} **DEPTH EXTENDER KIT** FOR THE **MAX PROTECTION POLYCARBONATE FACE SHIELD**

DEPTH EXTENDER KIT

SKU: PFSL-DEK

- 3-layers of cushioning provide additional clearance for longer surgical loupe models
- Place between forehead & shield
- Apply/detach with hook & loop attachment strips
- Each layer can be used separately or all together to provide additional depth from 3/8" up to 1"
- Wick tech sweat absorbent material
- *Compatible with models:*
 - Max Protection- Shield Only
 - Max Protection- Shield with Wick Guard


CONTENTS INCLUDES:

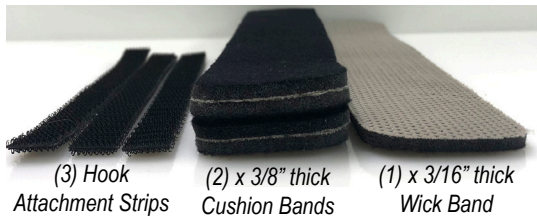

MAX PROTECTION SHIELD with WICK GUARD & DEPTH EXTENDER




MAX PROTECTION SHIELD ONLY & DEPTH EXTENDER

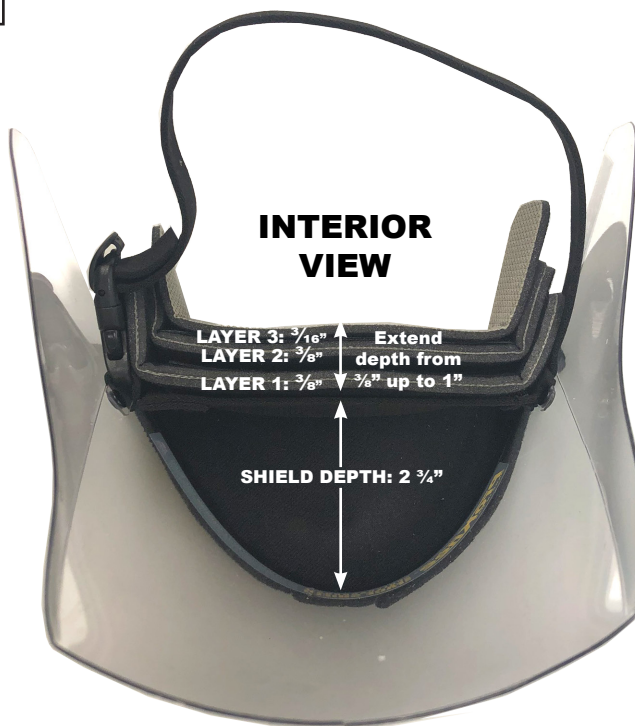

SETUP:

1 Place 1 x Hook Attachment Strip on 3/8" Cushion Band #1

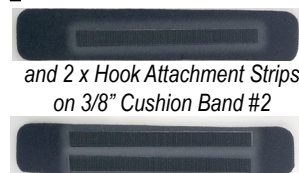

and 2 x Hook Attachment Strips on 3/8" Cushion Band #2

2 Stack Cushion Band #2 on TOP of Cushion Band #1



3 Stick stack of Cushion Bands to head strap



**If no Wick Guard, skip to step 5*

4 Stick Wick Guard to TOP Hook Attachment Strip



Stick 3/16" Wick Band to BOTTOM Hook Attachment Strip



ProPointer **FIT & DEPTH**

Our Max Protection shield depth is approximately 2.75" +/- deep when worn, but can vary by head circumference. We suggest **adjusting the head strap fit (by loosening the strap) and adjusting the head strap placement (up or down on your head).**

If loupes are still not clearing the shield, we offer a **DEPTH EXTENDER KIT** to provide up to 1" of additional shield depth. Multiple Depth Extender Kits can be combined to achieve additional depth.

4 DEPTH EXTENDING OPTIONS:

