

# ACCOMMODATE EVEN LONGER LOUPES WITH THE **ProKnee** DEPTH EXTENDER KIT FOR THE MAX PROTECTION POLYCARBONATE FACE SHIELD

**SINCE 1989**  
**MADE IN USA**

## DEPTH EXTENDER KIT

**SKU: PFSL-DEK**

- 3-layers of cushioning provide additional clearance for longer surgical loupe models
- Place between forehead & shield
- Apply/detach with hook & loop attachment strips
- Each layer can be used separately or all together to provide additional depth from 3/8" up to 1"
- Wick tech sweat absorbent material
- *Compatible with models:*
  - Max Protection- Shield Only
  - Max Protection- Shield with Wick Guard


### CONTENTS INCLUDES:



**MAX PROTECTION SHIELD with WICK GUARD & DEPTH EXTENDER**



**MAX PROTECTION SHIELD ONLY & DEPTH EXTENDER**

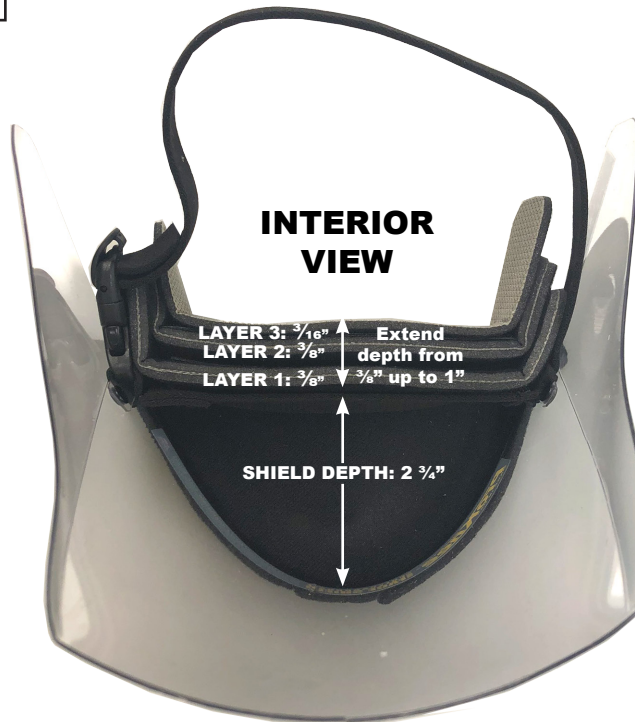

## SETUP:

**1** Place 1 x Hook Attachment Strip on 3/8" Cushion Band #1



and 2 x Hook Attachment Strips on 3/8" Cushion Band #2

**2** Stack Cushion Band #2 on TOP of Cushion Band #1



**3** Stick stack of Cushion Bands to head strap



*\*If no Wick Guard, skip to step 5*

**4** Stick Wick Guard to TOP Hook Attachment Strip



Stick 3/16" Wick Band to BOTTOM Hook Attachment Strip



## ProPointer → FIT & DEPTH

Our Max Protection shield depth is approximately 2.75"+/- deep when worn, but can vary by head circumference. We suggest adjusting the head strap fit (by loosening the strap) and adjusting the head strap placement (up or down on your head).

If loupes are still not clearing the shield, we offer a DEPTH EXTENDER KIT to provide up to 1" of additional shield depth. Multiple Depth Extender Kits can be combined to achieve additional depth.

### 4 DEPTH EXTENDING OPTIONS:

